

DAI-R™

Intramuscular Injection

 **SHL GROUP**
www.shl-group.com



Technical Specifications

Device Type	Disposable Autoinjector
Primary Container	1mL Glass Syringe Luer Compatible
Dosage Type	Fixed Full Contents
Needle Extension	Up to 1 1/4" or 31mm Intramuscular
Needle Insertion	Automatic
Needle Removal	Automatic
Activation Method	Button

Copyright © 2014 SHL Group. All rights reserved. Version 1.0 Information accurate at time of print and is subject to change without notice. Available as standard or customized to client specifications. Contact info@shl-group.com for more information.

Handling Instructions

1.



Remove sterile foil from needle

2.



Snap syringe cap off

3.



Attach needle. Do not remove needle cap

4.



Arm device by pulling needle cover. Needle cap will pop off automatically

5.



Press against injection site & push activation button

+



Remove device from injection site & discard in sharps container