

**PPI<sup>®</sup>***The ultimate in precision & accuracy*Volume Viscosity **Device Type**

Auto Injector

**Primary Container**

Pre-Filled 2.25mL Glass Syringe

**Dosage Type**Fixed | 10  $\mu$ L**Usage**

Multiple Dose (200 times) | Disposable

**Needle Insertion**

Manual

**Activation Method**

Button Activated

**Injection Feedback**

Audible click at beginning of injection

## Handling Instructions



1  
Screw needle onto the device to attach



2  
Twist the load knob clockwise until it comes to a stop



3  
Remove needle cap then press dose activator to prime



4  
Insert needle to desired depth & press dose activator to inject. Move to next injection site & repeat



+  
Rewind the device when delivery time exceeds 3 seconds

## Key Features

- ✓ Accurate & precise injection
- ✓ Audible, visual & tactile feedback
- ✓ Ergonomic & innovative design
- ✓ Delivers small dosages
- ✓ One-handed operation
- ✓ Accommodates high viscosity formulations